

Canine Protective Elbow Sleeve



Standard Length Adjustable DogLeggs

This easy-to-apply product provides coverage, padding and protection for the elbow joints. It is primarily used to treat and prevent elbow hygromas and is also commonly used to treat decubital ulcers, pressure sores, calluses (including pendulous calluses) and elbow arthritis.

For elbow hygromas, the use of the DogLeggs can be curative, eliminating the risk of complications associated with draining or surgery. For animals suffering from elbow arthritis, the padding offers protection as well as reflected body heat to the joint improving comfort and mobility.

Indications for Use:

-  Hygroma (preventative and curative)
-  Decubital ulcer
-  Pressure sore
-  Callus/pendulous callus
-  Lick granuloma
-  Elbow dysplasia
-  Elbow osteoarthritis
-  Skin lesions associated with radiation therapy
-  Post-operative bandage wounds
-  Abrasions and incisions




Specifications:

Standard Length Adjustable DogLeggs is constructed from a 3-dimensional fleece-faced, highly breathable, 4-way stretch textile with wicking properties. This material allows air to circulate while keeping the joint warm. The elbow cuffs contain 7mm of padding, sufficient for even giant breed dogs. VELCRO® Brand fasteners are used for adjustments and attachments. Product is machine washable.

How to measure:

DogLeggs are custom-sized to the measurements you provide. For accurate measurements, perform all measurements with your pet standing and use a dressmaker's flexible cloth or vinyl tape measure. Measure with enough tautness to dent the fur but not the skin.

- 1 Measure from the point of elbow on one side of your dog, over the back, to the point of the elbow on the opposite side of your dog.
- 2 Measure around your dog's left and right legs at the point of the elbow (these measurements may be different, especially if there is swelling).
- 3 Measure around your dog's left and right legs 4 inches below the point of the elbow (if the swelling extends below 4 inches, please contact our office as additional measurements may be needed). If your dog is small (point of elbow to floor is 6 inches or less), measure around each leg midway between the top of the paw and point of elbow.
- 4 Measure from the point of the elbow to the top of the paw.

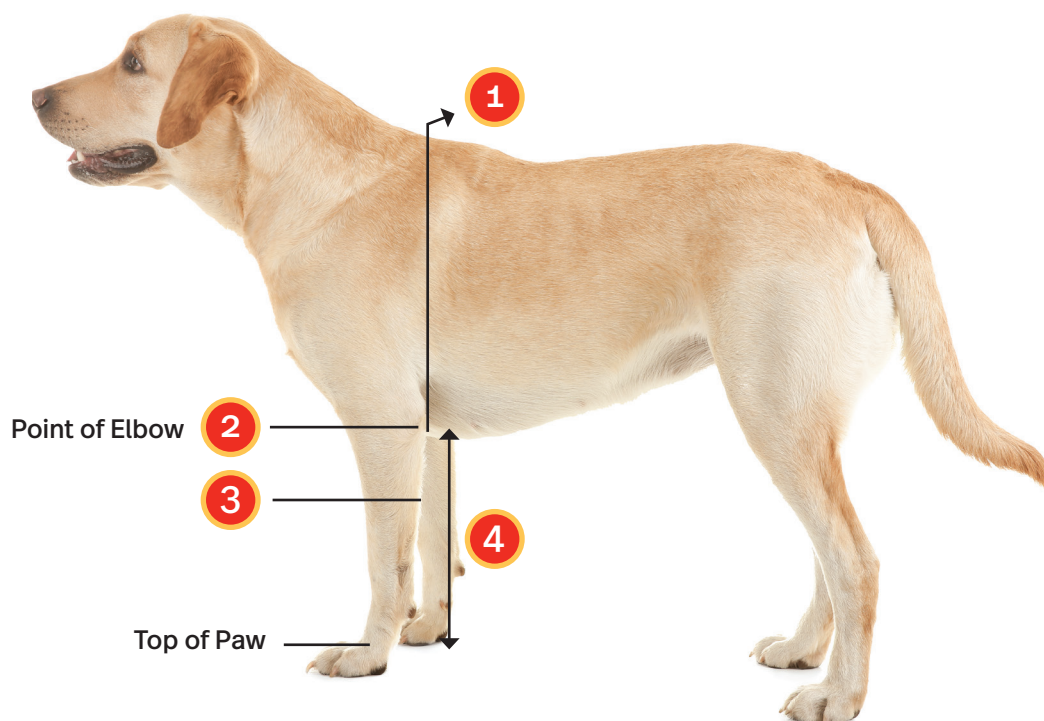


PATTERSON
VETERINARY

SKU #2300.99